

Affordable Automated, No Touch Airborne Decontamination

**Validated Efficacy
Bactericidal, Sporicidal,
Virucidal & Fungicidal
according to European standards**

**Ticking all the boxes for
Infection Control and
Decontamination procedures**



- ✓ Eliminates public enemies such as:
 - **MRSA**
 - **Clostridium difficile**
 - **Norovirus**
 - **Aspergillus niger**
- ✓ **Validated, proven efficacy** utilising hydrogen peroxide technology
- ✓ Safe access to treated areas from just **30 minutes!**
- ✓ **Weights only 6KG** – portable and versatile capacity of 10m³ to 1000m³
- ✓ Deployment – **easy, quick and safe to use.** no need to remove equipment from room
- ✓ **Low service and maintenance costs** proportionate and affordable.
- ✓ **Non corrosive, No residue.** no post-diffusion moisture or wipe down

Key areas of usage:

- Hospital wards, departments, operating theatres, side rooms, toilets & public areas
- Healthcare Transport and Public Transport
- Doctor and Dental surgery waiting rooms
 - Prison and Police custody cells
 - Veterinary practices
- Nursing homes and Residential care
- General Hospitality and Cruise Liners
 - Fire & flood restoration support
- Crime & trauma scene cleaning support

New

Nocolyse Food

Food area/manufacturing decontamination

Complimentary range of Insecticides

OXYPY:

eliminates flying or crawling insects & acarids including scabies

OXYPY+:

eliminates bed bugs and growth inhibitor of their larvae

Innotec Hygiene Solutions Ltd – UK Distributor for Oxypharm.
Contact us for more information or to arrange a demonstration

W: www.innotechygienesolutions.com

E: sales@innotechygienesolutions.com T: 07791797955

Oxypharm Machines



Nocospray

Used to treat areas from 10 to 1000m³



Nocomax Easy

Used to treat areas from 500 to 20,000m³



Nocomax DeLux

Similar design and volume to 'Easy' but with advanced technology features

Oxypharm range of disinfectants



Nocolyse Neutral

for preventative treatments (also available with mint fragrance)



Nocolyse One Shot

for curative treatments



Nocolyse Nocodor

for preventative treatments with essential oils to deodorise

Food area disinfection



Nocolyse Food

for preventative treatments of areas coming into contact with food

Oxypharm Insecticides



OXYPY - and - OXYPY+

for flying or crawling insects, acarids & scabies. Plus bespoke bed bug eradicator

Quality monitoring kit



Nocotest

Nocobox

Nocotest measures the presence of H₂O₂
Nocobox tests presence of microorganisms

Accessories



Room Decontamination in process
DO NOT ENTER
Call the staff to enter once notice has elapsed

PROCESS	START TIME	STOP TIME	OPERATION
Disinfection	08:00	09:00	Disinfection
Disinfection	10:00	11:00	Disinfection
Disinfection	12:00	13:00	Disinfection
Disinfection	14:00	15:00	Disinfection
Disinfection	16:00	17:00	Disinfection
Disinfection	18:00	19:00	Disinfection
Disinfection	20:00	21:00	Disinfection
Disinfection	22:00	23:00	Disinfection
Disinfection	00:00	01:00	Disinfection
Disinfection	02:00	03:00	Disinfection
Disinfection	04:00	05:00	Disinfection
Disinfection	06:00	07:00	Disinfection

Accessories include:

- Wall crossing system
- Door sealing tape
- Re-usable process warning sign



www.innotechygienesolutions.com



OXY'PHARM®



Nocospray

Developed around a heating and ionizing turbine that transforms and emits the Nocolyse range of disinfectants into a non-corrosive, non allergenic, biodegradable gas – with no residue.

The Nocospray and Nocolyse concept offers a new generation of fully automated, no touch, room decontamination based on hydrogen peroxide.

Also used for the diffusion of the Oxypharm range of Insecticides.

(see separate information sheets on chemical range and their purpose)

- **Lightweight (6KG)**
- **Portable**
- **Low purchase cost**
- **Quick process time**
- **Proven efficacy**
- **Low chemical consumption**
- **Easy to use**
- **Sensibly priced servicing**

Nocospray is available in three volume capacity options:

- 10–250m³
- 25-500m³
- 50–1000m³



Additionally Nocospray can be provided with:

- Remote Control
- Tracking System
- Timer Option

Any Nocospray device may also be produced in alternative colour of cover. A Silver or Ladybird design cover may be supplied for a supplementary charge.

Nocospray is available as:

- | | | |
|---|-------------------------------|-------------------------------|
| 10-250m³ – purchase code 3000.001. | with Remote Control 3003.001. | with Tracking System 3002.001 |
| 20-500m³ – purchase code 3000.002. | with Remote Control 3003.002. | with Tracking System 3002.002 |
| 50-1000m³ – purchase code 3000.003. | with Remote Control 3003.003. | with Tracking System 3002.003 |

NB: Tracking System Option includes Timer Option as standard.

Timer Option offered as alternative to Remote Control

Silver Cover Option – add code 3000.100 Ladybird Design Cover Option add code 3001.002



Innotec Hygiene Solutions Ltd. UK importer and distributor for Oxypharm.
www.innotechygienesolutions.com sales@innotechygienesolutions.com



OXY'PHARM®

Nocomax Easy



Developed around a heating and ionizing turbine that transforms and emits the Nocolyse range of disinfectants into a non-corrosive, non allergenic, biodegradable gas – with no residue.

The Nocomax Easy and Nocolyse concept offers a new generation of fully automated, no touch, area decontamination based on hydrogen peroxide.


Also used for the diffusion of the Oxypharm range of Insecticides.

(see separate information sheets on chemical range and their purpose)

- **Treats volumes between 500 and 20,000m³**
- Proven Efficacy
- Comparative low purchase cost
- Low chemical consumption (only 1ml per m³ for preventative treatment)
- Sensibly priced servicing
- Wheeled for ease of transportation



- LED indicates progression of treatment or malfunction
- Touchpad control
- Easy to programme
- Simple start/stop button procedure

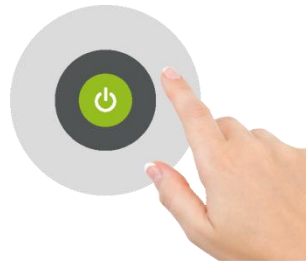

Nocomax Easy:

500-20,000m³ – purchase code 3100.100


To only be used with Nocolyse Disinfectants and OXPYPY and OXPYPY+ Insecticides – **20L containers**

Nocolyse Neutral 20L - 4000.020 Nocolyse Mint 20L - 4001.020 Nocolyse One Shot 20L - 4010.020

Nocolyse Nocodor 20L - 4030.020 OXPYPY 20L - 4200.020 OXPYPY+ 20L - 4230.020



Innotec Hygiene Solutions Ltd. UK importer and distributor for Oxypharm.
www.innotechygienesolutions.com sales@innotechygienesolutions.com

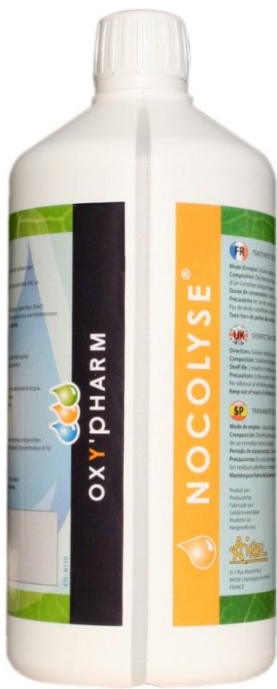

Nocolyse Neutral

A Bio-disinfectant for air and surfaces.

Nocolyse Neutral is a product based on hydrogen peroxide (6%), ready for use and produced to be used exclusively with the appliances **Nocospray** and **Nocomax**.


The combination of Nocolyse Neutral with Nocospray (or Nocomax) has **validated efficacy** on all types of microorganisms as a frequent use, **preventative** process. It is proven to be bactericidal, sporicidal, virucidal and fungicidal.

Nocolyse is also available with a mild Mint Fragrance – **Nocolyse Mint**.



Nocolyse must be used with the devices Nocospray, Nocomax or Nocomax Easy.

Achieving a good quality of disinfection is directly linked to a stringent cleaning protocol to be carried out before treatment. Its maintenance depends on a regular application of preventative treatments.

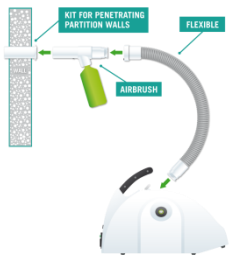

Nocolyse may also be used with our **Wall Cross System**. A method for disinfection from outside of treated area – ideal for scenarios where power may not be available (e.g. toilets) or to speed up multi-area applications (e.g. cruise ship cabins or ambulance fleet). The Wall Cross System comprises: – Wall cross inlet/cap (3010.110) 3m extension pipe (3010.000) and spray gun (3010.100).



Nocolyse Neutral is available in:

6 x 1L bottles for use with Nocospray – purchase code 4000.001-6

20L containers for use with Nocomax – purchase code 4000.020

Nocolyse Mint is available in:


6 x 1L bottles for use with Nocospray – purchase code 4001.001-6

20L containers for use with Nocomax – purchase code 4001.020





OXY'PHARM®

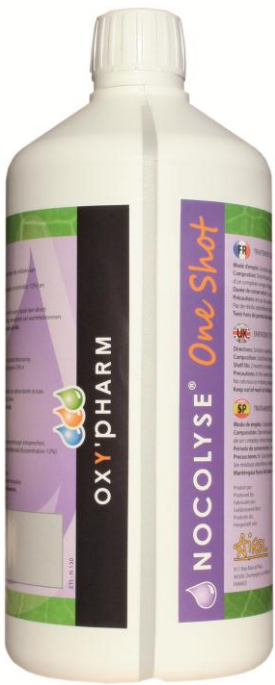

Nocolyse One Shot

A Bio-disinfectant for air and surfaces offering **reinforced disinfection** treatments.

Nocolyse One Shot is a product based on hydrogen peroxide (12%), ready for use and produced to be used exclusively with the appliances **Nocospray** and **Nocomax**.


The combination of Nocolyse One Shot with Nocospray (or Nocomax) has **validated efficacy** on all types of microorganisms as an occasional use, **Curative** process. It is proven to be bactericidal, sporicidal, virucidal and fungicidal.

One Shot must be used with the devices **Nocospray, Nocomax or Nocomax Easy.**

Achieving a good quality of disinfection is directly linked to a stringent cleaning protocol to be carried out before treatment. If regular preventative treatments cannot be carried out, or if there is a known infection present, it is essential for a curative process to be conducted.



One Shot may also be used with our **Wall Cross System**. A method for disinfection from outside of treated area – ideal for scenarios where power may not be available (e.g. toilets) or to speed up multi-area applications (e.g. cruise ship cabins or ambulance fleet). The Wall Cross System comprises: – Wall cross inlet/cap (3010.110) 3m extension pipe (3010.000) and spray gun (3010.100).

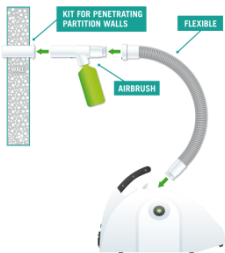


Nocolyse One Shot is available in:
6 x 1L bottles for use with Nocospray – purchase code 4010.001-6
20L containers for use with Nocomax – purchase code 4010.020



Innotec Hygiene Solutions Ltd. UK importer and distributor for Oxypharm.
www.innotechygienesolutions.com sales@innotechygienesolutions.com



OXY'PHARM®

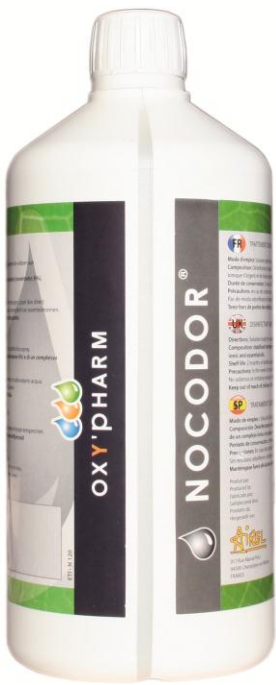

Nocolyse Nocodor

A **dual action** Bio-disinfectant and Deodoriser for air and surfaces.


Nocolyse Nocodor is a product based on hydrogen peroxide (6%) also comprising essential oils which have ethylene destruction properties. To be used exclusively with the appliances **Nocospray** and **Nocomax**.

The combination of Nocolyse Nocodor with Nocospray (or Nocomax) has **validated efficacy** on all types of microorganisms as a frequent use, **preventative** process. It is proven to be bactericidal, sporicidal, virucidal and fungicidal.



Nocodor must be used with the devices Nocospray, Nocomax or Nocomax Easy.

Achieving a good quality of disinfection is directly linked to a stringent cleaning protocol to be carried out before treatment. Its maintenance depends on a regular application of preventative treatments.

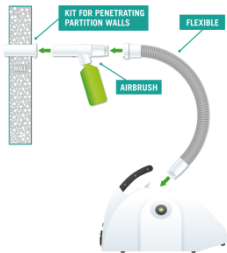

Nocodor may also be used with our **Wall Cross System**. A method for disinfection from outside of treated area – ideal for scenarios where power may not be available (e.g. toilets) or to speed up multi-area applications (e.g. cruise ship cabins or ambulance fleet). The Wall Cross System comprises: – Wall cross inlet/cap (3010.110) 3m extension pipe (3010.000) and spray gun (3010.100).



Nocolyse Nocodor is available in:

6 x 1L bottles for use with Nocospray – purchase code 4030.001-6
20L containers for use with Nocomax – purchase code 4030.020



Innotec Hygiene Solutions Ltd. UK importer and distributor for Oxypharm.
www.innotechygienesolutions.com sales@innotechygienesolutions.com



Nocolyse Food

An airborne Bio-disinfectant for surfaces coming into contact with **food**.


Nocolyse Food is a product based on hydrogen peroxide (8%), ready for use and produced to be used exclusively with the appliances **Nocospray** and **Nocomax**.

The combination of Nocolyse Food with Nocospray (or Nocomax) has **validated efficacy** on all types of microorganisms as a frequent use, **preventative** process. It is proven to be bactericidal, sporicidal, virucidal and fungicidal.



Nocolyse must be used with the devices Nocospray, Nocomax or Nocomax Easy.

Achieving a good quality of disinfection is directly linked to a stringent cleaning protocol to be carried out before treatment. Its maintenance depends on a regular application of preventative treatments.

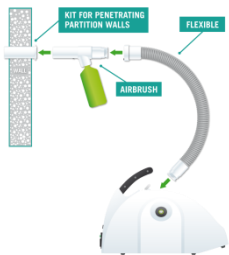

Nocolyse may also be used with our **Wall Cross System**. A method for disinfection from outside of treated area – ideal for scenarios where power or access may not be available or to speed up multi-area applications. The Wall Cross System comprises: – Wall cross inlet/cap (3010.110) 3m extension pipe (3010.000) and spray gun (3010.100).



Nocolyse Food is available in:

6 x 1L bottles for use with Nocospray – purchase code 4020.001-6
20L containers for use with Nocomax – purchase code 4020.020



OXYPY

Insecticide specifically developed for treatment against flying or crawling insects and acarids (including scabies).

A ready to use solution for internal use only.

OXYPY contains natural **pyrethrin** which delivers immediate **flash effect**. This quick action enables the destruction of all insects.

The natural pyrethrin is extracted from chrysanthemum flower heads and has **potent insecticidal properties**. Its flash effect kills every insect present at treatment time.

OXYPY also contains **Permethrin**. This is a common synthetic chemical widely used as an insecticide and is from the pyrethroid family of chemicals functioning as a neurotoxin. This targets the insects nervous system. Permethrin has a long lasting remnant effect of **4 weeks efficiency**.



OXYPY can be used with the Nocospray/Nocomax devices or can be applied manually using the spray head provided with each bottle.



It may also be used with our diffusion kit. A hand held application using Nocospray and accessories – extension shaft (3010.000) and spray gun (3010.100).



OXYPY is available in:

1L bottles for use with Nocospray and/or trigger spray – purchase code 4200.001

20L containers for use with Nocomax – purchase code 4200.020





OXY'PHARM®

OXYPY+

Insecticide specifically developed for treatment against bed bugs and their larvae.

A ready to use solution for internal use only.

OXYPY+ contains natural **Pyrethrin** which delivers immediate flash effect. This quick action enables the destruction of living bed bugs.



A catalytic effect is created by the presence of **Piperonyl Butoxide** - extracted from Sassafras wood - which reinforces the pyrethrin action.

S-Methoprene is added to the formula to deliver the larvicidal effect. This eradicates larva that will hatch from pyrethrin insensitive eggs. Its larvicidal action prevents larva from becoming adult. The larva is unable to reach stage IV of growth and create Chitin. It will therefore die within a week maximum without being able to become a blood sucking adult..

OXYPY+ can be used with the Nocospray/Nocomax devices or can be applied manually using the spray head provided with each bottle.



It may also be used with our diffusion kit. A hand held application using Nocospray and accessories – extension shaft (3010.000) and spray gun (3010.100)



OXYPY+ is available in:

1L bottles for use with Nocospray and/or trigger spray – purchase code 4230.001

20L containers for use with Nocomax – purchase code 4230.020



Innotec Hygiene Solutions Ltd. UK importer and distributor for Oxypharm.
www.innotechygienesolutions.com sales@innotechygienesolutions.com

Wall Crossing System

Enables the decontamination of a room or vehicle
without entering.

Ideal for treatments where there may be no power source in target area e.g. Toilet and bathrooms and prevents potential hazards of using extension cables.



The Wall Cross System comprises
3 component parts:

- Wall cross inlet/cap
- Spray gun
- 3m extension pipe


to be used with Nocospray and
any of the 1L bottles of chemicals.

Creates time for further treatments as the Nocospray can be moved and utilised immediately after diffusion rather than remaining in treatment area during the settlement phase.

Significant benefit in areas such as vehicular fleet and Cruise liner cabin decontamination.

Simple ease of application.

The cap on the inlet is clipped on after treatment which may be locked for security.

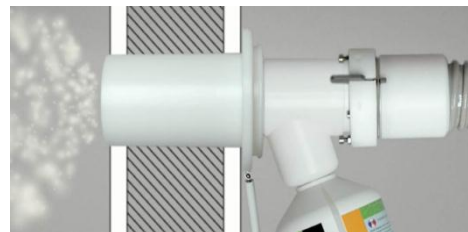

Wall cross inlet/cap: Purchase code 3010.110

Spray gun: Purchase code 3010.000

3M extension pipe: Purchase code 3010.000



OXY'PHARM®

Wall Mounted Nocospray

Enables the use of Nocospray as a fixed appliance.

Delivers the same airborne automated diffusion of decontamination products as the portable Nocospray but from a permanent wall siting.

Ideal for use in transport, single use applications where storage may be an issue or where plug in power source is not permitted or available (e.g. washroom/toilet areas).

Wall Mounted Nocospray is available in two versions:

- Mounted on the wall of the target treatment room – as pictured above
- External wall mounted with wall crossing system – as pictured below



The Wall Mounted Nocospray with **Wall Crossing System** is fixed to the wall outside the target treatment area and diffused via the wall cross hose and through to target area.

Designed for use in some medical vehicles where internal and external wall are separated with a cavity in which the device is placed.

Both Wall Mounted Nocospray's are available in two volume capacity options:

10–250m³ & 50–1000m³

Wall Mounted Nocospray is available as:

10-250m³ – purchase code 3009.061. **50-1000m³** – purchase code 3009.063.

Wall Mounted Nocospray with Wall Cross System is available as:

10-250m³ – purchase code 3009.071. **50-1000m³** – purchase code 3009.073.



Innotec Hygiene Solutions Ltd. UK importer and distributor for Oxypharm.
www.innotechygienesolutions.com sales@innotechygienesolutions.com

Quality Monitoring Kit

Nocotest

Nocotest is a tube of 100 colorimetric strips that detect the presence of hydrogen peroxide.

Nocotest strips should be placed in a room prior to treatment in a variety of areas and their placement noted.

The detection tabs on one end of the strip will change colour in the presence of hydrogen peroxide.

The colour change informs that the diffusion was successful and evenly distributed within the room.



Nocobox

Nocobox are Petri dish contact plates with an agar medium to detect the presence of microorganisms

Nocobox should be used to take samples from surfaces both prior to, and after treatment.

Once left to culture, Nocobox may be observed for results after 48 hours



Nocotest – Tube of 100 strips: Purchase code 5000.010
 Nocobox – Box of 10 testing plates: Purchase code 5000.020

